

Silicone and wax wetting agent

500ml bottle with pump spray bottle

Application:

Shake well before use!

Close bottle tightly after use!

Apply by means of the pump spray bottle.

Spray silicone moulds sparingly and evenly before pouring with plaster, as well as wax objects and wax patterns before investing. Keep a distance of the pump spray point to the object of approx. 15 – 20 cm. Rotate object while spraying. The use of Silicone and wax wetting agent facilitates high precision reproduction of wax work into castings. Sharp edges, marginal areas, fissures, crowns insides are reproduced with high acutance and without micro bubbles. Small retentions and clasps of partial frameworks are reproduced in high fidelity.

General protection and hygienic measures:

- Keep away from food and beverage.
- Wash the skin thoroughly after work and before breaks.
- Keep away from flames and ignition springs.
- Avoid contact with eyes and skin, do not inhale vapours.

Inhalation:

Provide for fresh air. Use a convenient face mask.

Hand protection:

Wear PVC safety gloves

Eye protection:

Wear safety goggles

Body protection:

Wear antistatic clothing made of natural fibres (e.g. cotton)

Properties:

Easily flammable

Storage:

When used and stored properly no decomposition will occur

Observe the regulations for the storage of flammable liquids

Storage time max. 2 years

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/14