# Li-Bond

**DFS**  
Dental Future Systems  
**D I A M O N**

Ländenstraße 1 | 93339 Riedenburg  
Phone +49 (0)9442-9189-0  
info@dfs-diamon.de | www.dfs-diamon.de

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Light curing resin for modeling bridges, inlays, onlays, castable patterns & fixing attachments, extremely fast curing with LED light. Easy to trim when cured, Shore-Hardness D 80-85. Temperature range – 40°C to + 150°C, burns out without residue.

#### Components

**Pen Kit #26600** consists of 6 volt curing light, cartridge & jig (holds the cartridge & light)

**Refill Cartridge Pack #26602:** Pack of 2 flexible cartridges with fine metal tip for accurate & detailed application. Store in dark for up to 12 months shelf life.

**LED Light (6 volt) #26603:** Blue light LED of specially modified wave length. Can be used with or without cartridge jig.

#### Uses & Application Instructions

Fast & efficient method for connecting many laboratory materials including wax, metal, resins, plaster & stone. Ideal as a stable base layer for telescopic crowns, connecting attachments, implant-abutments. Adheres well to wax, can be used to connect complex wax bridge-work sections. Temporary fixing of attachments & telescopic crowns for soldering. Quick & effective fixing or stabilizing for soldering or denture repairs

#### Safety guidelines

- ✗ Never look into the light directly
- ✗ Avoid material contact with skin or eyes
- ✗ After contact with skin wash thoroughly with water
- ✗ After contact with eyes rinse thoroughly with water and seek medical advice
- ✗ Li-Bond is not intended as a permanent fixative
- ✗ Li-Bond is not indicated for intra oral use
- ✗ Close cartridge immediately after use quickly and store in dark conditions