

Clean-Orange

Application:

Surface cleaning of furniture and stainless steel surfaces in dental clinics and labs. Cleaning of impression trays. Acrylic surfaces may show slight damage when using Clean-Orange.

Mode of Application:

Mix with water depending on the intensity of dirt to be cleaned.

Quality:

Removes residue of silicone, alginates, zinc-eugenol pastes and blood. Not suitable for use in ultrasonic cleaners.

Risks:

Do not drink or swallow, please consult a doctor if the liquid is swallowed. Avoid contact to skin and eyes, may cause skin irritations.

Disposal:

Do not pour into the drain the pure condition.

Manufacturer:

DFS-DIAMON GmbH * Ländenstraße 1 * D-93339 Riedenburg * +49 9442 91890

Rev. 11/14